SARS-CoV-2 Rapid Antibody Test

REF	$\overline{\mathbb{V}}$	SYSTEM
9901-NCOV-02C	40	visual reading

English Intended use The SARS-CoV-2 Rapid Antibody Test is a rapid chromatographic immunoassay intended for the qualitative in vitro detection of antibodies to SARS-CoV-2 in human serum, plasma or whole blood. The SARS-CoV-2 Rapid Antibody Test is intended for use as an id in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating prior infection. The SARS-CoV-2 Rapid Antibody Test is intended for professional use in laboratory and point-diverse anyinomments.

Summary Coronaviruses are single-stranded positive-sense RNA viruses with an envelope of about 80 to Coronaviruses are single-stranded positive-sense RNA viruses with an envelope of about 80 to 120 nm in diameter. Its genetic material is the largest of all RNA viruses and is an important pathogen of many domestic animals, pets and humans. They can cause a variety of acute and chronic diseases. Common signs of a person infected with a coronavirus include respiratory symptoms, fever, cough, shortness of breath, and dyspnea. In more server cases, infection can cause pneumonia, severe acute respiratory syndrome, kidney failure, and even death. The 2019 and a pandemic was declared by the World Health Organization on March 11, 2020. WHO confirmed that COVID-19 can cause colds and more serious diseases such as severe acute respiratory syndrome (SARS). The test results are for clinical reference only and cannot be used alone as a basis for confirming or excluding cases.

Test principle The SARS-CoV-2 Rapid Antibody Test has three pre-coated lines, "C" control line, "G" and "M" The SARS-CoV-2 Rapid Antibody Test has three pre-coated lines, "C" control line, "G" and "M" test line for the device on the surface of the nitrocellulose membrane. The control line and two test lines in the result window are not visible before applying any samples. Monoclonal chicken lgY antibody is coated on the control line region and monoclonal anti-human IgG antibody and monoclonal anti-human IgM antibody are coated on the "G" and "M" test line region. And anti-chicken IgY antibody conjugated with colloidal gold particles are used as detectors for "C" control line. During the test, SARS-CoV-2 specific antibodies in the sample interact with recombinant SARS-CoV-2 protein conjugated with colloidal gold particles making antibody-antigen gold particle complex. This complex migrates on the membrane via capillary action until the "M" and "G" test line, where it will be captured by the monoclonal anti-human IgG antibody-anti-human IgM antiborus A colored test line would be visible in the same line and "G" test line, where it will be captured by the monoclonal anti-human IgG antibody-antibudy antibudy antibudy antibudy and construction sample interact. antibody or monoclonal anti-human IgM antibody. A colored test line would be visible in the antibody or monocional anti-human igM antibody. A colored test line would be visible in the result window if SARS-CoV-2 specific antibodies are present in the sample. The intensity of the colored test line will vary depending upon the amount SARS-CoV-2 antibodies present in the sample. If SARS-CoV-2 specific antibodies are not present in the sample, then no color appears in the test line. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents of the control line are working.

Reagents

- Monoclonal anti-human IgM
- Monoclonal anti-human IgG Chicken IgY
- Monoclonal anti-SARS-CoV-2 nucleocapsid protein
- Recombinant SARS-CoV-2 nucleocapsid protein gold conjugate
- Recombinant SARS-CoV-2 spike protein gold conjugate

Monoclonal anti-chicken IgY

Precautions and warnings This kit contains components classified as follows in accordance with the Regulation (EC)

No 1272/2008



H317 May cause an allergic skin reaction

H412 Harmful to aquatic life with long lasting effects.

Prevention P261

- Avoid breathing dust/fume/gas/mist/vapours/spray
- P273 Avoid release to the environment.

P280 Wear eye protection/ face protection

- Respons P333 + P313 If skin irritation or rash occurs: Get medical advice/attention
- P362 + P364 Take off contaminated clothing and wash it before reuse

Disposal: P501

Dispose of contents/container to an approved waste disposal plant. Product safety labeling follows EU GHS guidance. Contact phone: all countries: +49-621-7590

- Do not re-use the test kit.
- Do not use the test kit if the pouch is damaged or the seal is broken. Do not use the buffer of different lot.
- Do not smoke, drink or eat while handling sample
- Wear personal protective equipment, such as gloves and lab coats when handling kit reagents. Wash hands thoroughly after the tests are done.
- Clean up spills thoroughly using an appropriate disinfectant.
- Handle all samples as if they contain infectious agents.
- Observe established precautions against microbiological hazards throughout testing procedures
- Dispose of all samples and materials used to perform the test as biohazard waste. Laboratory chemical and biohazard wastes must be handled and discarded in accordance with all local,
- state, and national regulations. Desiccant in foil pouch is to absorb moisture and keep humidity from affecting products. If the moisture indicating desiccant beads change from vellow to green, the test device in the pouch
- should be discarded. Good laboratory practice recommends the use of the control materials. Users should follow the appropriate federal state, and local guidelines concerning the frequency of assaying
- external quality control materials. For in vitro diagnostic use

Exterior a quarty comor materizes. For in vitro diagnostic use. Exercise the normal precautions required for handling all laboratory reagents. Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

Storage and stability Store the kit at room femperature, 2-30 °C/36-86 °F, out of direct sunlight. Kit materials are stable until the expiration date printed on the outer box. Do not freeze the kit. The test must be used within 1 hour once the pouch has been opened.

Sample collection and preparation Carefully read these instructions and also the enclosed Quick Reference Guide (with illustrations) before using the SARS-CoV-2 Rapid Antibody Test.

1. Collect the whole blood into the commercially available plain tube, NOT containing anti-coagulants such as Sodium heparin, K2-EDTA, Sodium citrate by venipuncture and leave to settle for 30 minutes for blood coagulation and then centrifuge blood to get serum sample of supernatant.

2. If serum in the plain tube is stored in a refrigerator at 2-8 °C/36-46 °F, the sample can be used for testing within 1 week after collection. Using the sample in the long-term keeping more than 1 week can cause non-specific reaction 3. They should be brought to room temperature prior to use

3 Not for the screening of donated blood

determined by this qualitative test.

performance and/or produce invalid results.

infection.

strictly when testing.

Specific performance data

7-14 days after symptom onse

> 14 days after symptom onset

Sensitivity

Sensitivity

Specificit

before December 2019. All sample results were negative.

Analytical performance Limit of detection: SN titer 1:40

SARS-CoV-2 Rapid Antibody Test

SARS-CoV-2

SARS-CoV-2

Rapid Antibody Test

Interfering substance

Zanamivir (Influenza)

Artemether-lumefantrine (Malaria drug)

Quinine (Malaria drug)

Ribavirin (HCV drug)

Tenofovir (HIV drug)

Tinidazole

Praziguantel

Rapid Antibody Test

Limitations

4. The test procedure should be conducted in ambient temperature and pressure.

3. Test results must be considered with other clinical data available to the physician

6. A negative result may occur if the concentration of antibody in a sample is below the

9. For diagnostic purposes, the results should always be assessed in conjunction with the

Clinical evaluation Performance characteristics for the SARS-CoV-2 Rapid Antibody Test for rapid detection of Performance characteristics for the SARS-CoV-2 Rapid Antibody Test for rapid detection of

Performance characteristics for the SAHS-CoV-2 Hapid Antibody lest for rapid detection of anti-SAHS-CoV-2 antibodies were established in retrospective, multi institutes, randomized, single-blinded study conducted at a trial site in Korea during the 2020 SARS-CoV-2 pandemix

single-blinded study conducted at a trial site in Korea during the 2020 SAHS-CoV-2 pandemic situation. A total of 379 retrospective samples were tested using the SARS-CoV-2 Rapid Antibody Test. These samples consisted of serum from PCR positive or negative confirmed patients. The performance of the SARS-CoV-2 Rapid Antibody Test was compared to a commercialized molecular assay. Although the SARS-CoV-2 Rapid Antibody Test allows to test for IgM and IgG separately, due to the differing inter-patient time response to the virus, any individual with positive result for the IgM or the IgG test should be read as a positive for anti-SARS-CoV-2 antibodies. The combined test result (positive for IgM and/or IgG or negative for IgM and/or IgG) was used to calculate the total test sensitivity and specificity.

In an altor ingo was used to carculate the rotal rest sensitivity and specificity. • Test sensitivity The seroconversion time of IgM and IgG antibodies varies from person to person, but it was estimated to be around 7-14 days after onset of symptoms.¹² The SARS-CoV-2 Rapid Antibody Test showed a sensitivity of **92.59** % compared to PCR in samples taken 7-14 days after symptom onset in SARS-CoV-2 PCR positive patients.

The SARS-CoV-2 Rapid Antibody Test showed a sensitivity of 99.03 % compared to PCR in samples taken > 14 days after symptom onset in SARS-CoV-2 PCR positive patients.

Test specificity
 The SARS-CoV-2 Rapid Antibody Test showed a specificity of 98.65 % compared to PCR in
 samples from SARS-CoV-2 PCR negative patients.

Limit of detection: SN titer 1:40 Cross-reactivity: No cross-reactivity for the specimen of HIV Seroconversion panel, Japanese Encephalitis positive, Zika virus positive, Chikungunya positive, Dengue IgM positive, Salmonella typhi IgM positive, Treponema pallidum, HAV IgM positive, HAV IgG positive, HBV Ab positive, HCV Ab positive, Influenza vaccine positive, Leishmania positive, Brouella IgM positive, Chagas positive, Toxoplasma positive, Finariasis positive, RSV positive, Haremophilis Influenza positive, Mycoplasma pneumonia IgM positive, Mycoplasma pneumonia IgG positive, Influenza A IgM positive, Influenza B IgM positive, Influenza A and B IgG-IgM positive, Tuberculosis positive, non-SARS-CoV of MERS CoV or coronaviruses HKU1, NL63, OC43 and 229E. But, the SARS-CoV-2 Rapid Antibody Test may cross-react with antibody against SARS-CoV. Additional cross-reactivity test results: In a separate evaluation, 18 samples from a cohort with common cold were tested with the SARS-CoV-2 Rapid Antibody Test for potential cross-reactivity. The samples were obtained before December 2019. All sample results were negative.

Interference study: All tested interfering materials do not affect sensitivity and specificity of the SARS-CoV-2 Rapid Antibody Test (test concentrations according to CLSI EP07-A2). The SARS-CoV-2 Rapid Antibody Test was not affected by interfering materials at the following concentrations:

terfering substance

Oseltamivir (Influenza)

Lamivudine (Retroviral

Daclatasvir (HCV drug)

nedication)

Metronidazole

Niclosamide

Rifampicin

Doxycycline hyclate (Malaria

Interfering

5 mg/mL

50 µM

150 uM

1 mg/mL

1 mg/mL

1 mg/mL

1 mg/mL

Positive

Negative

Total

Positive

Negative

Total

Positive

Negative

Total

Positive

50

54

Positive

102

103

Positive

0

0

PCB

Negative

0

0

0

(50/54, 95 % Cl. 82,11 % -97,94 %)

PCR

Negative

0

0

99.03 %

(102/103, 95 % Cl. 94,71 % -99,98 %)

PCR

Negative

219

222

98 65 %

(219/222, 95 % Cl, 96.10 %-99.72 %)

92.59

Total

50

4

54

Total

102

1

103

Total

3

219

222

Interfering

10 mg/mL

mg/mL

mg/mL

701 µmol/L

1mg/mL

80 µmol/L

70 uM

level

6. The SARS-CoV-2 Rapid Antibody Test should not be used to diagnose acute SARS-CoV-2

1. The test procedure, precautions and interpretation of results for this test must be followed

2. This test detects the presence of SARS-CoV-2 IgM/IgG in the sample and should not be used

4. Neither the quantitative value nor the ratio anti-SARS-CoV-2 IgM/IgG concentration can be

5. Failure to follow the test procedure and interpretation of test results may adversely affect test

detection limit of the test or if the sample was collected or transported improperly, therefore a negative test result does not eliminate the possibility of SARS-CoV-2 infection, and should be

5. Results of these tests should be appropriately recorded in a test report.

as the sole criteria for the diagnosis of SARS-CoV-2 infection.

irmed by viral culture or an molecular assay or ELISA.

patient's medical history, clinical examination and other results.

7. Positive test results do not rule out co-infections with other pathogens

8. Negative test results are not intended to rule out other coronavirus infection.

- 1. Collect the venous blood into the commercially available anti-coagulant tube such as Sodium heparin, K2-EDTA, Sodium citrate by venipuncture and centrifuge blood to get plasma
- 2. If plasma in an anti-coagulant tube is stored in a refrigerator at 2-8 °C/36-46 °F, the sample can be used for testing within 1 week after collection. Using the sample in the long-term keeping more than 1 week can cause non-specific reaction.
- 3. They should be brought to room temperature prior to use.

Whole blood

- Capillary whole blood
- . Capillary whole blood should be collected aseptically by fingertip.
- 2. Clean the area to be lanced with an alcohol swab.
- 3. Refer to the instructions for use that come with the lancing device. Squeeze the end of the fingertip and pierce with a sterile lancet.
- 4. Using a capillary tube, collect the 20 µl of capillary whole blood to the black line of the capillary tube.
- The capillary whole blood must be tested immediately after collection.
- Venous whole blood
- Collect the venous whole blood into the commercially available anti-coagulant tube such as Sodium heparin, K2-EDTA, Sodium citrate by venipuncture.
- 2. If venous whole blood in an anti-coagulant tube is stored in a refrigerator at 2-8 °C/36-46 °F, the sample can be used for testing within 1-2 days after collection
- 3. Do not use hemolyzed blood samples.
- A Use separate disposable materials for each sample in order to avoid cross-contamination which can cause erroneous results.
- Materials provided Test device (individually in a foil pouch with desiccant)

Test device (individually in a foil pouch with desiccant) Buffer bottle Capillary tube (20 μ) Film (can be attached to the test device when performing outdoor testing) Instructions for Use Quick Reference Guide

Materials required (but not provided) Single use disposable lancing device (e.g. Accu-Chek Safe-T-Pro Plus) General laboratory equipment (e.g., sample transfer pipette for venous blood or alcohol wipes for the fingerstick

- Test procedure Carefully read these instructions and also the enclosed Quick Reference Guide (with illustrations) before using the SARS-CoV-2 Rapid Antibody Test.
- 1. Check the expiry date at the back of the foil pouch. Do not use the test device, if expiry date has passed.
- 2. Open the foil pouch and remove the test device and the desiccant package.
- 3. Check both the test device and the desiccant in the foil pouch. Ensure that the test device is undamaged and that the desiccant status indicator shows valid (yellow).
- Perform a QC as required according to the Instructions for Use of the QC material Performing a test with capillary whole blood
- 1. Clean a fingertip by wiping with an alcohol swab.
- 2. Refer to the instructions for use that come with the lancing device. Squeeze the end of the fingertip and pierce with a sterile lancet.
- 3. Collecting of sample: Using a capillary tube, collect the 20 µl of capillary whole blood to the black line of the capillary tube.
- 4. Adding of sample: Apply the collected capillary whole blood to the sample well of the test
- device. 5. Dropping of buffer: Add 3 drops (90 µl) of buffer vertically into the sample well of the test
- device. Add the buffer immediately after applying the sample.

 Reading time: Read the test result at 10-15 minutes. ∧ Bisk of incorrect results. Do not read test results after 15 minutes. It may give false results.

Performing a test with serum, plasma, or venous whole blood

1. Collecting of sample: Using a micropipette, collect the 10 µl of serum, plasma or 20 µl of venous whole blood with micropipette.

- 2. Adding of sample: Add the collected serum, plasma or venous whole blood to the sample well of the test device.
- 3. Dropping of buffer: Add 3 drops (90 µl) of buffer vertically into the sample well of the test device. Add the buffer immediately after applying the sample.

4. Reading time: Read the test result at 10-15 minutes. ☆ Risk of incorrect results. Do not read test results after 15 minutes. It may give false results.

Understanding results

- 1. A colored band will appear in the top section of the result window to show that the test is working properly. This band is the control line (C). 2. Colored bands will appear in the lower section of the result window. These bands are the test
- lines of IgM/IgG (M, G). 3 Even if the control line is faint or the test line isn't uniform the test should be considered to
- be performed properly and the test result should be interpreted as a positive result.
- Results from antibody testing should not be used as the sole basis to diagnose or exclude SARS-CoV-2 infection or to inform infection status.
- Positive results should be considered in conjunction with the clinical history, RT-PCR results and other data available.

as required by test procedures in this instructions and in accordance with local,

1. Negative results do not rule out SARS-CoV-2 infection, particularly in those who have been in

contact with the virus. Follow-up testing with a molecular diagnostic should be considered to

2. Results from antibody testing should not be used as the sole basis to diagnose or exclude

once for each new lot

once for each untrained operator

Notification for SARS-CoV-2 Antibody Tests

SARS-CoV-2 infection or to inform infection status

rule out infection in these individuals

- 1. Positive and negative controls are optional contents (SARS-CoV-2 Antibody Control
- Cat No. 99COVC20) and these controls can be provided as a means on additional QC to demonstrate a positive or negative reaction. 2. Quality controls should be treated and tested the same as patient samples.

3. It is recommended that positive and negative controls be run:

state and federal regulations or accreditation requirements.

Interfering level
200 µM
3 mM
82 µM
130 µM
310 µM
1200 ng/mL
100 µg/mL
5 ng/mL
3.4 µmol/L
3.8 % (w/v)
Commercial kit titer > 1:1280
1.794 mg/dL
Commercial kit O.D 4.17
120 mg/mL

terfering substance

Ervthromvcin (antibiotic)

Bilirubin (Conjugated)

Anti-Nuclear Antibody (AN

Human antibodies to E.coli

Serum total proteins

cetaminophe

Ibuprofer

Kanamvcin

Cholesterol

Sodium citrat

Elevated IgG

Caffeine

Biotin

FDTA

nterfering

300 µmol/L

4 mM

350 uM

31 µM

60 µM

90 mM

90 mM

15 ng/mL

200 mg/mL

3000 U/L

ter 802

3480 IU/mL

229 mg/dL

60 ma/mL

3 CLSI EP07-A2 / Vol. 25 No. 27, Interference Testing in Clinical Chemistry.

Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim

T Promedt Consulting GmbH Altenhofstrasse 80 66386 St. Ingbert Germany

Suwon-si, Gyeonggi-do, 16690 REPUBLIC OF KOREA

There was no interference with serum of pregnant women

SD BIOSENSOF

www.sdbiosensor.com

Phone : +49 6894 581020 Fax : +49 6894 581021

Roche order number: 09216448190

EC REP Authorized Bepresentative

Interfering substance

Acetylsalicylic acid

Ciprofloxacin (antibiotic)

Bilirubin (Unconjugated)

Human anti-mouse antibod

L-ascorbic acid

Isoniazid

Ampicillir

Ethanol

Trialvcerides

Hemoglobin

Rheumatoid factor

Human serum albumin

K2-EDTA-treated tubes)

Elevated IgM

Matrix equivalen

()

Heparin

The matrix and anticoagulants do not affect the detection of SARS-CoV-2 IgG and IgM in contrived specimens of serum, plasma (Sodium heparin, K2-EDTA, Sodium citrate), venous whole blood (Sodium heparin, K2-EDTA, Sodium citrate), and capillary whole blood (collected in

 References
 1
 Guo L et al. Profiling Early Humoral Response to Diagnose Novel Coronavirus Disease (COVID-19). Clinical Infectious Disease. 2020.

 2
 Zhao J et al. Antibody responses to SARS-CoV-2 in patients of novel coronavirus disease 2019. Clinical Infectious Disease. 2020.

lead office: C-4th&5th, 16, Deogyeong-daero 1556beon-gil, Yeongtong-gu, Manufacturing site: 74, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu Cheongju-si, Chungcheongbuk-do, 28161 REPUBLIC OF KOREA