



# GCE Zen-O lite™

## Portable Oxygen Concentrator

The Zen-O lite™ from GCE is a powerful and robust portable oxygen solution that opens the doors to a wide range of users that want to maintain an active lifestyle.

With 5 pulse flow settings available and weighing just 5.5lbs (2.5kg), the Zen-O lite™ provides a happy medium between those wanting a compact and lightweight solution together with the capability of delivering oxygen at a more demanding level.



Pulse Flow

**1 to 5**  
Pulse flow  
settings



Weight

**5.5lbs**  
(2.5kgs)



Battery life

Up to  
**8**  
Hours  
on setting 2\*



*intermedical*  
HOMECARE


[www.portableoxygen.co.uk](http://www.portableoxygen.co.uk) | 0800 028 2194

Trusted in Respiratory Health

# The little powerful portable oxygen solution.

The control panel features easy to understand intuitive buttons and LCD display for straight-forward operation. Oxygen is delivered across 5 pulse flow settings, adjustable in 0.5 increments. The machine responds to the user's respiration rate by automatically adjusting the amount of oxygen delivered with every breath.

The Zen-O lite™ is easily worn over the shoulder with the provided protective carry bag. A rucksack accessory can also be used (sold separately) for even greater comfort whilst mobile. Packages on offer include one or two battery packs. Each battery pack lasts up to 4 hours and slots into the bottom of the machine. Desktop charger also available (sold with double battery package or separately).

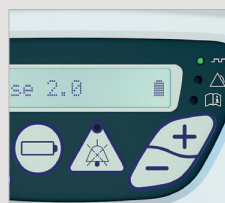


▲ The optional rucksack accessory allows for the machine to be worn over both shoulders.

▲ The display shows current pulse flow setting and power source (battery or mains). Pushing the battery indicator button will display remaining operation time in % and other battery information. The breath detection indicator (to the right of the display) flashes every time a breath is detected giving users assurance oxygen is being delivered. The pulse flow settings can be adjusted in 0.5 increments using the + and - buttons on the control panel.

## Single Battery Package

- Zen-O lite™ main unit
- Rechargeable battery pack
- AC power supply unit
- DC power supply unit
- Carry bag
- User manual
- Nasal cannula

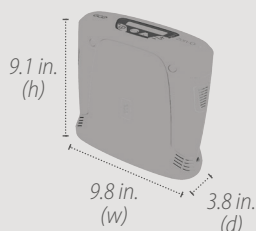

## Double Battery Package

- As above including:
- Additional battery pack
  - Desktop battery charger

23.1 (h) x 24.9 (w) x 9.7 (d)  
in cm

## Approximate battery run-time (in hours:minutes)

	1 x battery pack	2 x battery packs
Setting 1	4:00	8:00
Setting 2	4:00	8:00
Setting 3	3:00	6:00
Setting 4	2:15	4:30
Setting 5	2:00	4:00
*Run-time dependent on breathing rate		
Concentration	90% -3% /+ 6%	
Weight	5.5lbs (2.5kg) with battery pack installed. Battery pack: 1lb (0.45kg)	

Power	AC: 100-240v DC: 11.5-16v
Noise	38db(A) on setting 2
Batt. charge time	Approx. 3 hours
Warranty	3 years on unit 1 year on batteries, sieve beds & accessories
Airline approved	Yes, subject to airline's terms & conditions



This device may be suitable for use whilst sleeping. This device cannot be used with a CPAP or BiPAP device.

**intermedical**  
HOMECARE

0800 028 2194  
[www.portableoxygen.co.uk](http://www.portableoxygen.co.uk)

Intermedical (UK) Limited, Homecare Division  
Unit 6 Mill Hall Business Estate, Aylesford, Kent, ME20 7JZ, United Kingdom



Cert: 87M18032